

INSTRUCTION FOR USE / EN

Additional language versions of the instructions for use are available on our website www.olident.com or on request at: info@olident.com

OliEtchBOND

PRODUCT INFORMATION

OliEtchBOND is a simple to use self-etching light curing one component adhesive. It is designed for strong bonding of composites to enamel and dentin.

OliEtchBOND is compatible with all current brands of visible light cure composite restorative materials.

INDICATIONS

Self-etching light cure adhesive.

CAVITY PREPARATION

Rubber dam is the recommended method of isolation.

Clean the tooth with flour of pumice and water prior to preparation. Prepare the cavity with minimal tooth reduction. Margins should have a slight (0.5 - 1.0 mm) bevel placed in the enamel to increase the surface area for greater bond strength.

PULP PROTECTION

Cavity floor of deep excavations should be covered with a thin layer of calciumhydroxide material.

APPLICATION

Apply OliEtchBOND with a brush onto the moist enamel and dentin surfaces for 30 seconds with agitation. The material should build a homogeneous layer. Air thin gently for 10 seconds to remove the volatile components and to disperse the adhesive. Then light cure with a suitable dental light unit for 20 seconds before placement of a composite.

Apply the restorative material according to the instructions of the manufacturer. Best results are obtained with application of a thin layer of a light cure flowable composite followed by the application of a moldable composite. Light cure each composite layer separately according to the corresponding user instructions.

STORAGE

Do not store above 25°C (78°F)! Avoid storage in direct sunlight. Do not use after expiration date. Keep away from children! For dental use only!

PACKAGING AND REFERENCE NUMBER

1x 5 ml bottle, this instruction for use	OL003E
--	--------

Reservation:

Olident Co. Ltd., L. P. warrants this product will be free from defects in material and manufacture. Olident Co. Ltd., L. P. makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application.

If this product is defective within the warranty period, your exclusively remedy and Olident Co. Ltd., L. P.'s sole obligation shall be repair or replacement of the Olident Co. Ltd., L. P. product.

Limitation of liability - except where prohibited by law, Olident Co. Ltd., L. P. will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability. The CE 0197 mark does not apply to accessories.

 **OLIDENT Sp. z o. o., Sp. k.**

ul. Podłęże 653
32-003 Podłęże
tel.: +48 690 449 304
info@olident.com

www.olident.com
www.olidentDENTALshop.pl



Made in EU

