

INSTRUCTION FOR USE / EN

Additional language versions of the instructions for use are available on our website www.olident.com or on request at: info@olident.com

OliEtch

PRODUCT INFORMATION

OliEtch is a 38% phosphoric acid for enamel and dentine etching.

COMPOSITION

Phosphoric Acid < 38%, inorganic filler, water, pigment.

APPLICATION

1. Remove cap of syringe.
2. Place a disposable applicator nozzle firmly onto gel syringe.
3. Make sure area to be etched is rinsed, clean and dry.
4. Apply gel to the required area.
5. Etching time: approximately 20 – 30 seconds for enamel. Maximum 15 seconds on dentine. After etching has taken place rinse surface thoroughly with water and vacuum to remove gel.
6. Dry with clean dry air.
7. The area will have a chalky white appearance.
8. Remove applicator nozzle from syringe, clean and replace cap.

STORAGE

Store at 4-25°C (39-77°F). If temperature exceeds this, storing in refrigerator is recommended.

Caution: do not freeze, do not expose to direct light or other heat sources. Return to room temperature before use.

LOT NUMBER AND EXPIRY DATE

The above are printed on each pack-age and should be quoted in all correspondence which requires identification of the product.

PRECAUTIONS

Use of protective gloves and a no-touch technique is recommended. If material contacts skin wash immediately with soap and water. When in con-tact with eyes rinse with plenty of water and seek medical advice.
Ingestion - seek immediate medical attention.

CONTRAINDICATIONS

Application of the above materials is contraindicated, if a dry working field cannot be established and/or if a patient is known to be allergic to any of the ingredients.
Keep away from children! For dental use only!

PACKAGING AND THE REFERENCE NUMBERS

1 x 3 ml syringe, 3 mixing tips	OL104 H 3
3 x 3 ml syringes, 9 mixing tips	OL104 SET

Reservation:

Ocident Co. Ltd., L. P. warrants this product will be free from defects in material and manufacture. Ocident Co. Ltd., L. P. makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product

for user's application. If this product is defective within the warranty period, your exclusively remedy and Olident Co. Ltd., L. P.'s sole obligation shall be repair or replacement of the Olident Co. Ltd., L. P. product.

Limitation of liability - except where prohibited by law, Olident Co. Ltd., L. P. will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability. The CE 0197 mark does not apply to accessories.



 **OLIDENT Sp. z o. o., Sp. k.**

ul. Podłęże 653
32-003 Podłęże
tel.: +48 690 449 304
info@olident.com

www.olident.com
www.olidentDENTALshop.pl



Made in EU

