




OliBULK Fill

FLOWABLE POSTERIOR COMPOSITE

Simply simple



OlibULK Fill IS A FLOWABLE COMPOSITE MATERIAL for direct fillings in permanent and deciduous posterior teeth using the Bulk Fill technique. Thanks to the optimal ratio of flowability to elasticity, it is quick and convenient to apply and shape.

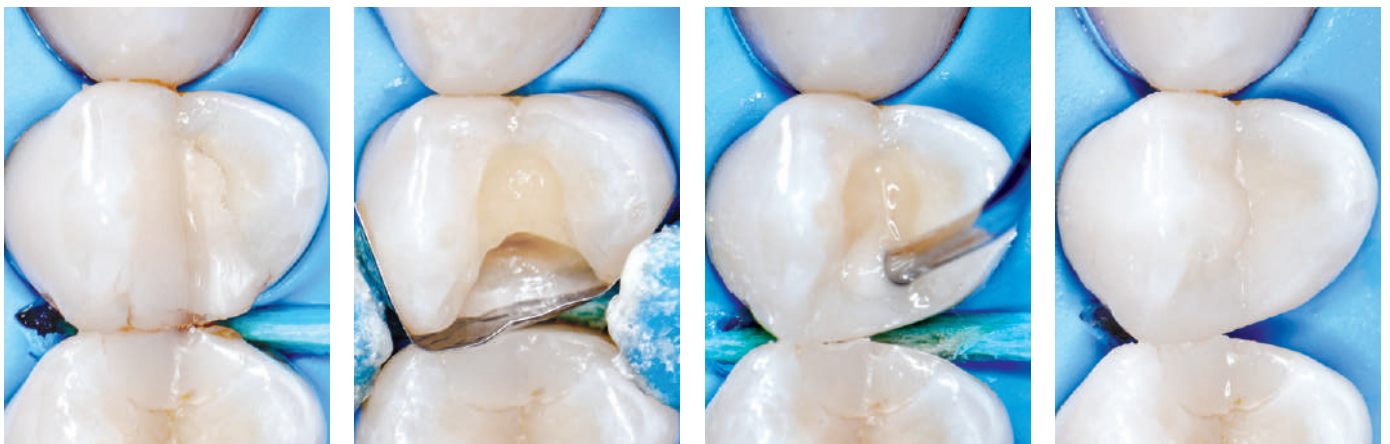

Secondary caries in tooth 24. Prepared cavity filled with OlibULK Fill UD. Cusps restored with OliREVO.
dr Karol Babiński, Poland, 2019.

choice

versatile use of your **choice**



OliBULK Fill
single layer technique



OliBULK Fill - core layer
eg. **VIP Posterior** - top layer



standard layering technique

OliBULK Fill is intended for single-step reconstructions with the omission of the layer technique. If needed though, it can also be used as the core layer and then covered with the mouldable posterior composite. A special monomer matrix ensures that the material adapts perfectly to cavity walls, even with the presence of undercuts.

aesthetic

aesthetic chameleon effect



- Prepared cavity.



- Final restoration.
OliBULK Fill, shade UD.

AVAILABLE SHADES:

UD



MD




- UD OliBULK Fill - Universal Dentin
MD OliBULK Fill - Medium Dentin

OliBULK Fill is available only in two all-purpose shades, because the clearly discernible „chameleon“ effect significantly affects the aesthetic qualities of the final restoration. The ease of polishing and the long-lasting gloss are additional advantages of this material.

Perfect

Perfect for:

- fillings with layer thicknesses up to 4 mm in Black's class I and II cavities (class II requires capping with the standard composite)
- cavity lining - as a first layer in Black's class I and II cavities
- deciduous teeth fillings



OliBULK Fill
up to 4 mm
in ONE layer

FEATURES

- layer of up to 4 mm
- two dentin shades with „chameleon” effect
- optimal ratio of flowability to elasticity
- excellent physical and mechanical properties
- high radiopacity (210% Al)
- Bis-GMA free

ADVANTAGES

- effective results
- esthetic finish
- perfect adaptation to the cavity walls
- high quality long-term fillings
- perfect visual control
- totally biocompatible




SCAN & WATCH

Product instructional video is available on our YouTube channel. Facts, tips and tricks are right here. For You.

LOW SHRINKAGE STRESS





OliBULK Fill is characterized by low polymerisation shrinkage, which reduces the shrinkage stress. The tooth walls are safe and the fillings are not exposed to marginal leakage.

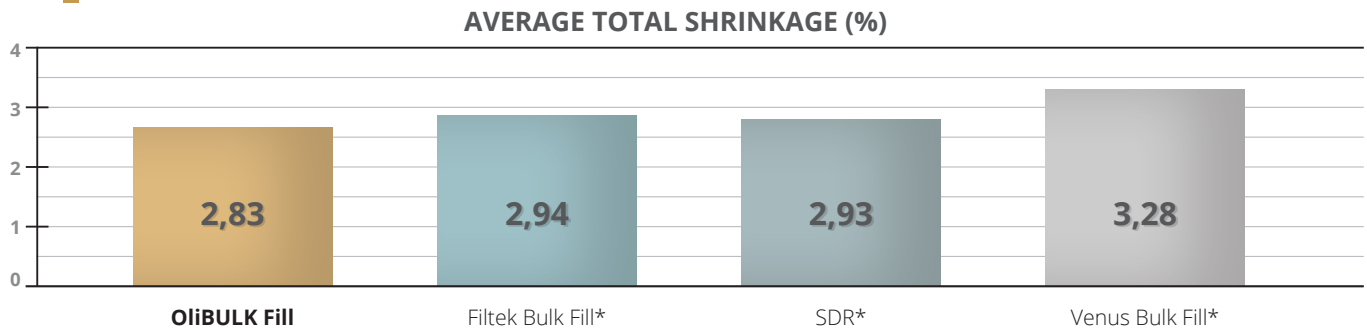

Source: Independent InnoMat GmbH, Teltow, Germany, 2012.

EXCEPTIONAL MECHANICAL PROPERTIES

The physical and mechanical properties of OliBULK Fill guarantee that using the singlelayer technique in extensive Class I cavities is no longer a compromise. The restorations are durable and resistant to high biting forces. Thanks to such a convenient and fast procedure, OliBULK Fill is the ideal material for extensive and irregularly shaped fillings.

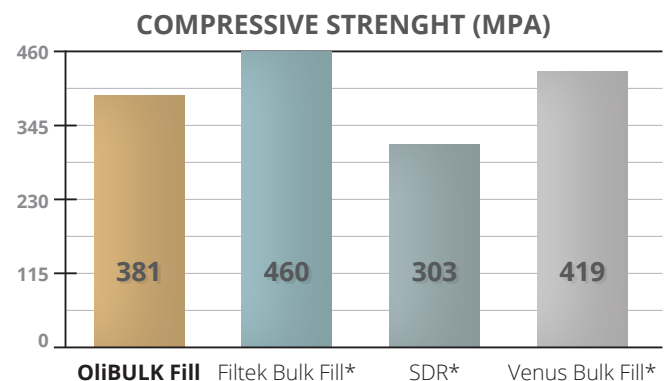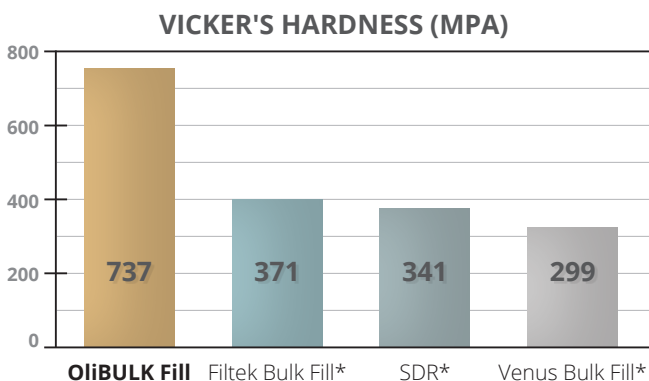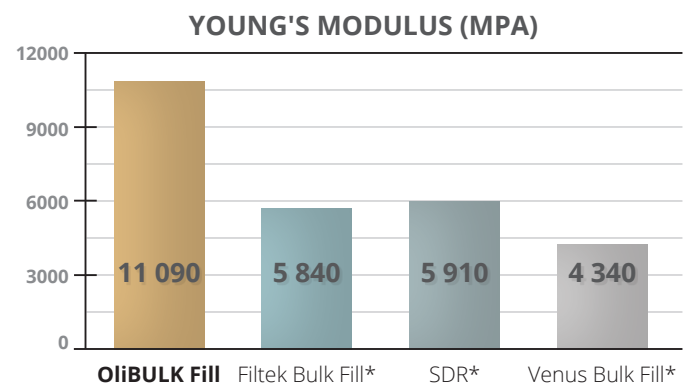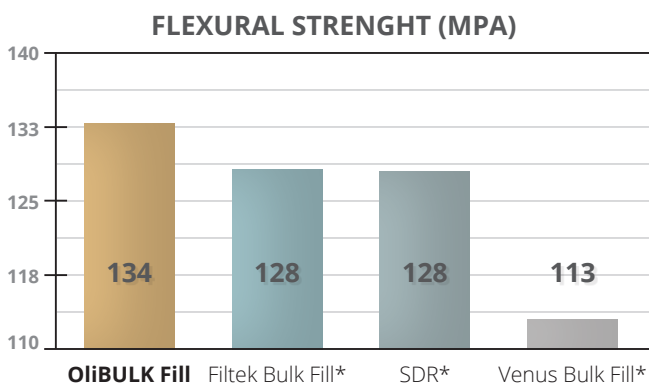


Source: Independent Fraunhofer Institut für Keramische Technologien und Systeme, Dresden, Germany, 2012.

* This brands aren't registered trademarks of Olident.

PROCEDURE

RECOMMENDED PROCEDURE AND COMPATIBLE PRODUCTS

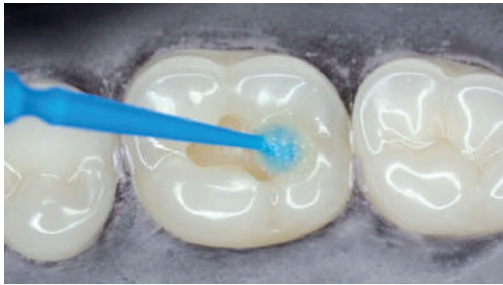


01

OliEtch BOND

- self-etching
- self-adhesive
- strong adhesion

Cavity treated with the self-adhesive bonding system shortens the procedure significantly.

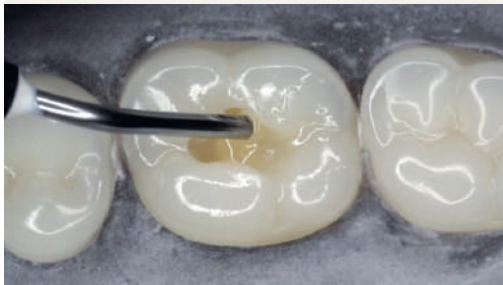

02

OliBULK Fill

- 4 mm layer
- low marginal stress
- resistant and durable

One-layer technique saves time and effort. Final effect is reliable and natural looking.

*An alternative stage of the procedure.
NOT used in this case!*

2.5

VIP Posterior

- more filler
- perfect consistency
- greater hardness

Designed for posterior teeth exposed to occlusal loads. Perfect for molar cusps shaping.



04

OliFinisher C132

- fin-less top
- profiled blades
- high quality shank

Specially profiled blades collect or smooth out the composite material and just glide over natural tissues.



05

Uporal D 603

- diamond coated
- sterilisation safe
- for multiple use

Unique diamond coating guarantees perfect polishing and gloss finish. Hence, polishing with felt can be skipped.

AVAILABLE PACKAGING



OliEtch BOND

5 ml bottle
OL003



OliBULK Fill

2 g syringe

UD / Universal Dentin
OL095

MD / Medium Dentin
OL117

OliBULK Fill

20 x 0,25 g capsules

UD / Universal Dentin
OL096

MD / Medium Dentin
OL118



VIP Posterior

5 g syringe

OL252 - A2
OL253 - A3
OL254 - A3.5



OliFinisher C132

2 pcs

C132.314.008 - 8 blades
C132F.314.008 - 16 blades
C132UF.314.008 - 30 blades



Uporal D 603

1 pc
code: **603**



OLIDENT[®]
INNOVATION

www.olident.com



ver. EN1/2019