EN INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION

OliTemp C is an automic non-eugenol temporary crown and bridge cement suitable for cementing temporary crowns or bridges as well as for trial cementing of permanent crowns and bridges. It shows excellent retention to temporary crown and bridge materials, but it can easily removed from prepared tooth stumps. **OliTemp C** is delivered in MINIMIX syringes so it is very easy and comfortable to use. Due to its non-eugenol formulation it does not influence the setting of resin-based filling and luting materials. The temporary crown and bridge cement is based on ZnO/organic acids. Especially in cases of allergies to eugenol it is strongly recommended to use this eugenol-free type of cement.

OliTemp C meets the requirements of DIN EN ISO 3107, type I and the ADA/ANSI Specification No. 30.

INDICATIONS/INTENDED USE

- Temporary cementing of provisional crowns or bridges
- Trial cementing of permanent crowns and bridges

PERFORMANCE FEATURES

The performance features of the product meet the requirements of the intended use.

CONTRAINDICATIONS

Do not use OliTemp C in cases where known allergies to the components exist.

PATIENT TARGET GROUP

Persons who are treated during a dental procedure.

INTENDED USERS

This medical device should only be used by a professionally trained dental practitioner.

APPLICATION

Preparing the MINIMIX-Syringe

Remove the cap of the MINIMIX-syringe and throw it away (do not use it again!). It is replaced by a special 1:1 mixing cannula. Turn the cannula 90° until it locks in position. The material is now ready for application.

Note: Store used syringe with fixed used mixing cannula. Discard the first 2-3 mm of the extruded material. This must be done for each new mix.

Precautions

OliTemp C should extrude easily from the MINIMIX-syringe. Do not use excessive force. Excessive pressure may result in an uncontrolled extrusion of the material and may cause rupture of the syringe.

Cementing

OliTemp C can be applied directly into the temporary crown or bridge under slight pressure from the mixing cannula.

Apply a thin layer to the inside of the temporary crown or bridge and seat not later than 1 minute (working time at 23 °C / 73 °F) cautiously but firmly over the prepared tooth structures.

Note: Do not dry stumps, best moisten them with some water. A humid milieu (humid stumps) additionally facilitates to remove the temporary crown or bridge later on.

OliTemp C sets in the mouth within 2-3 minutes. After OliTemp C has set remove any excess.

REMOVING OF THE RESTORATION

Remove the restoration with a suitable instrument. Residual OliTemp C has to be removed completely with a suitable instrument (e.g. scaler, explorer or hand instrument before final cementing).

Clean dentine and enamel with a moist cotton pellet. Wash thoroughly with water spray and blot dry.

STORAGE

Store at 8 – 25 °C (47 – 77 °F). Protect from moisture! Do not use after expiry date.

ADDITIONAL INFORMATIONS/WARNINGS

In case of contact with skin and eyes rinse immediately with plenty of water and consult a physician if necessary. If swallowed seek medical advice.

Keep away from children!

DISPOSAL

Disposal of the product according to local authority regulations.

REPORTING OBLIGATION

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

WARRANTY

SanaPro Dental GmbH warrants this product will be free from defects in material and manufacture. SanaPro Dental GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, you exclusively remedy and SanaPro Dental GmbH sole obligation shall be repair or replacement of the SanaPro Dental GmbH product.

LIMITATION OF LIABILITY

except where prohibited by law, SanaPro Dental GmbH. will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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SanaPro Dental GmbH, Stresemannstrasse 46